CHROMagar™ ESBL

For overnight detection of Gram-negative bacteria producing extended spectrum beta-lactamase.

This product is composed of the powder base CHROMagar Orientation (RT412 or RT413-25) and CHROMagar ESBL supplement (ES372 or ES373-25)

article number ESRT2 : 5000 ml Kit
(composed of RT412 + ES372)
article number ESRT3-25 : 25 L Kit
(composed of RT413-25 + ES373-25)

STORAGE

CHROMagar Orientation: Store the powder at 15/30°C until the shelflife date indicated on the label.

CHROMagar ESBL supplement: Store the powder at 2/8°C until the shelflife date indicated on the label.

COMPOSITION in g/L

CHROMagar Orientation: Agar 15,0; Peptone and yeast extract 17,0; Chromogenic mix 1,0; pH: 7,0 +/- 0,2 (Classical formula adjusted and/or supplemented as required to meet performance criteria).

CHROMagar ESBL supplement: Selective mix 0,57 g/L (Classical formula adjusted and/or supplemented as required to meet performance criteria).

PREPARATION

STEP 1:

Preparation of the powder base CHROMagar Orientation

1. Suspend the medium in the proportion of 33,0 g/L of purified water.
2. Disperse powder slowly in water by rotating for swelling of the agar.
3. Heat and bring to boiling (100°C) while swirling or stirring regularly and AUTOCLAVE at 121°C during 15 min.

For the 100°C heating step, mixture may also be brought to a boil in a microwave oven: after initial boiling, remove from oven, stir gently, then return to oven for short repeated bursts of heating until complete fusion of the agar grains has taken place (large bubbles replacing foam).

4. Cool in a water bath to 45-50°C, swirling or stirring gently.

STEP 2:

Prepare and integrate CHROMagar ESBL supplement (ES372 or ES373-25) to the melted base:

5. Aliquote the required supplement powder for 570 mg/L final. Add purified water to make a supplement solution at 57 mg/ml.

Warning 1: This step may require several minutes of stirring to obtain a good and homogenous suspension: opaque-yellowish appearance.

Warning 2: Reconstituted supplement solution must be used the same day. Do not store and re-use a supplement solution.

6. Vortex this supplement to homogenise and add, in the proportion of 10 ml/L of final media, to melted CHROMagar™ Orientation cooled at 45/50°C, then stir to make CHROMagar™ ESBL.

Examples of calculation:

Qsf 1 L media: Rehydrate 570 mg into 10 ml of purified water
Qsf 5 L media: Rehydrate 2850 mg into 50 ml of purified water

7. Pour into sterile Petri dishes or tubes and allow to gel and dry.

8. Store in the dark. Prepared media plates can be kept for one day at ambient temperature. Plates can be stored for up to two weeks under refrigeration (2/8°C) if properly prepared and protected from light and dehydration.

INOCULATION

If the agar plate has been refrigerated, allow to warm to room temperature before inoculation. Streak sample onto plate and incubate at 37°C for 18-24 hours.

INTERPRETATION

Microorganism → Typical colony appearance

Suspected ESBL strain
E.coli → dark pink to reddish
Klebsiella, Enterobacter, Citrobacter → metallic blue
Proteus → brown halo

Non Resistant Other Gram(-) strains → inhibited
Gram(+) strains → inhibited
Yeasts → mostly inhibited

PERFORMANCE and LIMITATIONS

- Final identification requires additional testing.
- Some *Pseudomonas sp* and *Acinetobacter sp*, widely-known to be frequently Multi Drug Resistant bacteria, could grow on the medium with typical colony aspects as expected on CHROMagar™ Orientation.

DISPOSAL OF WASTE After interpretation all plates should be destroyed by autoclaving at 121°C for at least 20 minutes.

English. For *In vitro* Diagnostic use. Laboratory product to be used only by trained personnel.

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CHROMagar™ and Rambach™ are trademarks of Dr. A. Rambach