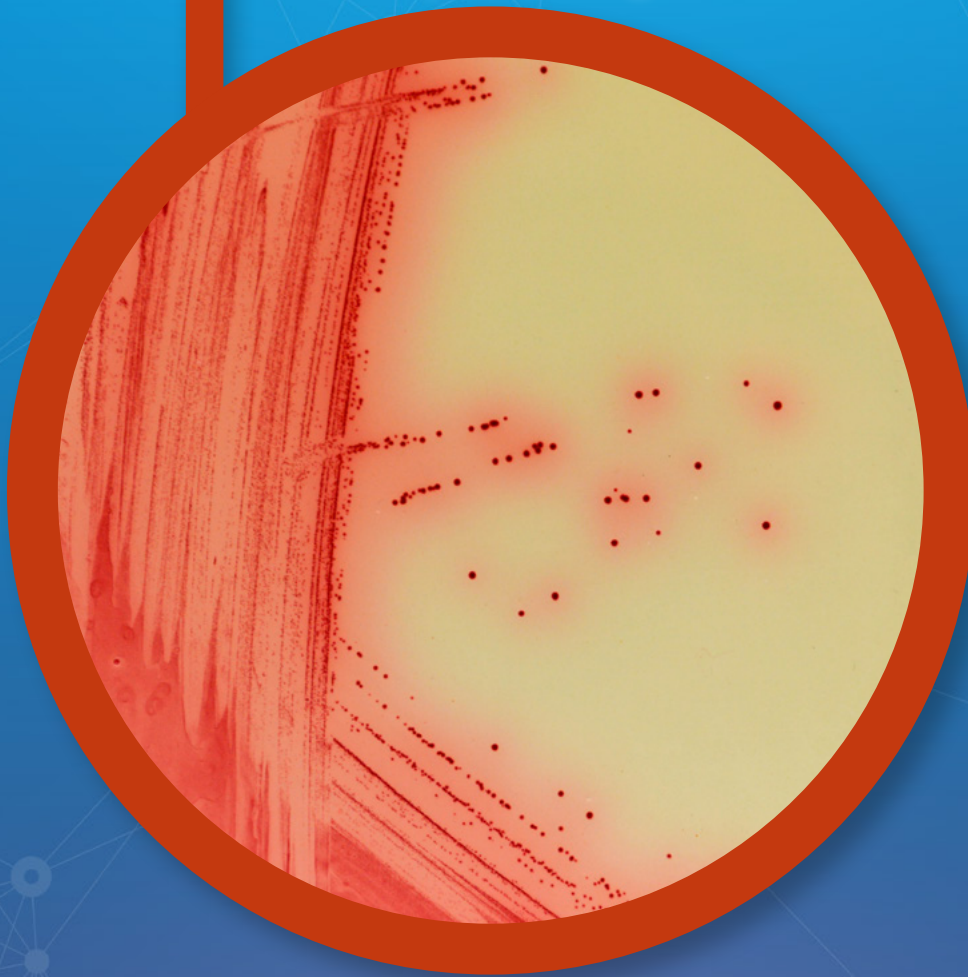
CHROMagar™ Mycoplasma

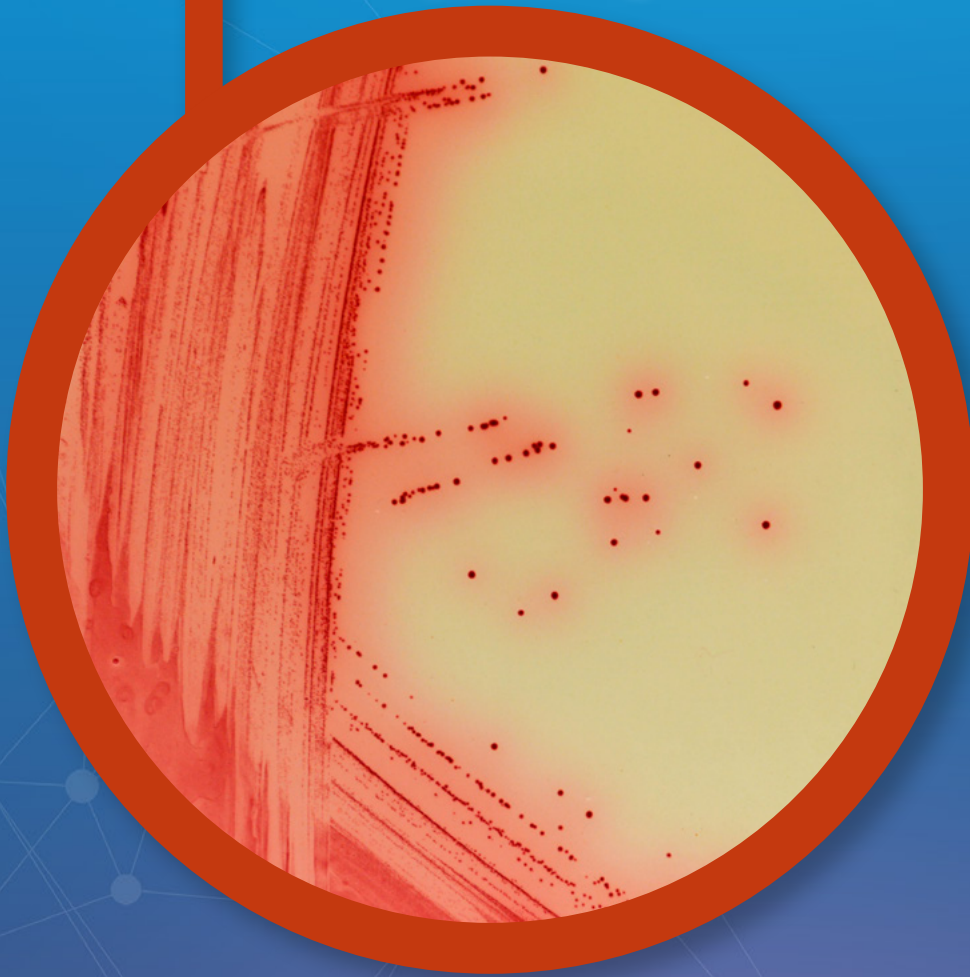
Click below:

EN

FR

ES

● CHROMagar™ Mycoplasma

For detection of *Mycoplasma bovis*

For detection of *Mycoplasma bovis*

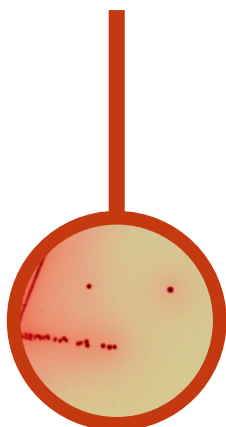
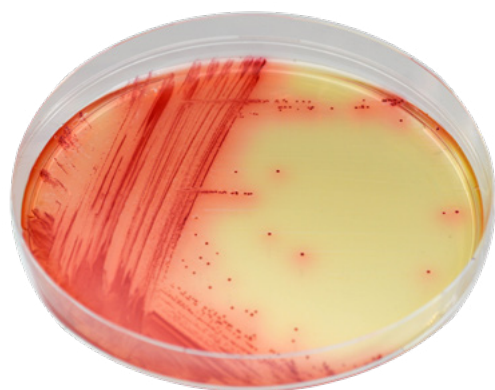
Plate Reading

- *Mycoplasma bovis*
→ red
- *Mycoplasma bovis*
under binocular loupe
→ red with a fried egg
morphology

Intended Use

Bovine infections can result in a variety of clinical signs, such as pneumonia mainly but also arthritis, mastitis and keratoconjunctivitis, none of which are *M. bovis*-specific. Laboratory diagnosis is therefore important.

Identification of *M. bovis* from nose swabs, broncho-alveolar lavage samples can be done by streaking the sample directly onto CHROMagar™ Mycoplasma agar plates.

The detection of *M. bovis* by red coloration allows an easy detection with a fried egg aspect under binocular loupe after 3-7 days of incubation at 37 °C under CO₂ atmosphere.

Medium Performance

- 1 EASY TO READ AND INTERPRET**
With CHROMagar™ Mycoplasma, *Mycoplasma* colonies have an intense red coloration and a fried egg appearance - making detection very easy.
- 2 DETECTION**
Detection is possible between 3 to 7 days under CO₂ atmosphere.
- 3 SIMPLE METHOD**
Identification of *M. bovis* from nasal swabs and broncho-alveolar lavage samples can be performed by plating the sample directly onto the plates.

Medium Description

Powder Base	Total 43.8 g/L Agar 10.0 Peptone and yeast extract 25.0 Mineral Salts 6.3 Growth factors 2.2 Chromogenic and selective mix 0.3 Storage at 15/30 °C - pH: 7.6 +/- 0.2 Shelf Life > 18 months
Supplement (S1) included in this reference	Powder 1 g/L Storage at 15/30 °C Shelf Life > 18 months
Supplement (S2) included in this reference	Powder 1 g/L Storage at 2/8 °C Shelf Life > 18 months
Supplement (S3) included in this reference	Lyophilized 150 mL/L Storage at 2/8 °C Shelf Life > 12 months
Usual Samples	Nose swabs and broncho-alveolar lavage samples
Procedure	Direct streaking. Incubate at 37 °C for 3 to 7 days under CO ₂ atmosphere.
Scientific Publications on this product: available on www.CHROMagar.com Please read carefully the instructions for use (IFU document) available on : www.CHROMagar.com	

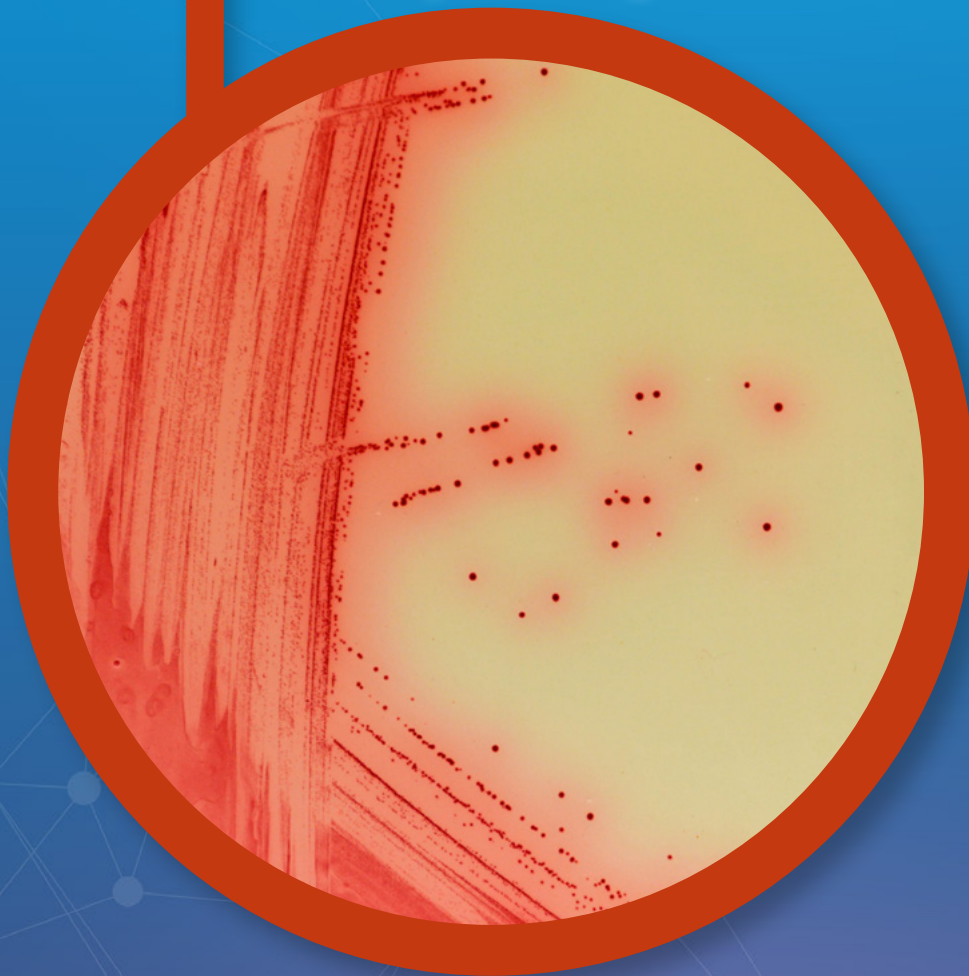
Order References

Please use these product references when contacting your local distributor:

1000 mL pack MB640
(Included in this reference: powder base MB640(B) + supplement MB640(S1) + supplement MB640(S2)
+ supplement MB640(S3))

Manufacturer: CHROMagar, 29 avenue George Sand, 93210 La Plaine Saint-Denis - France
Email: CHROMagar@CHROMagar.com
Website: www.CHROMagar.com
Find your nearest distributor on
www.CHROMagar.com/contact

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